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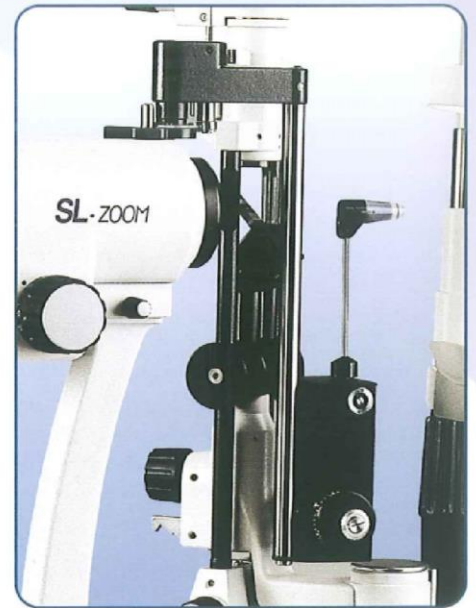
The applanation tonometers F 900 and A 900 are accessory appliances to the slit lamp SL 990 and they are used to measure ocular pressure. Thanks to its versatility the applanation tonometer can also be mounted and combined with other manufacturers equipment.

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The applanation tonometer operates according to the "Goldmann method". In other words, it measures the pressure required for maintaining uniform applanation of the corneal surface. Precise measurement of the small flattened area is made using a slit lamp at 10x magnification.

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Advantages

- Measurement at the patient's endocular pressure is made using a slit lamp, together with other routine microscopic measurements.
- Measurement accuracy is extremely high, the standard deviation among single measurements is approximately $\pm 0,5$ mmHg.
- The value is expressed in mmHg and it's read directly on the instrument.
- Scleral rigidity need not be taken into consideration because the small volume moved ($0,56$ mm^3) increases intra-ocular pressure by only about 2,5%.
- Repetition of the measurement procedure does not reduce ocular pressure since no "massage effect" has been reported.
- There are no difficulties about standardization and calibration.

Particolare manopola graduata
Graduated knob detail

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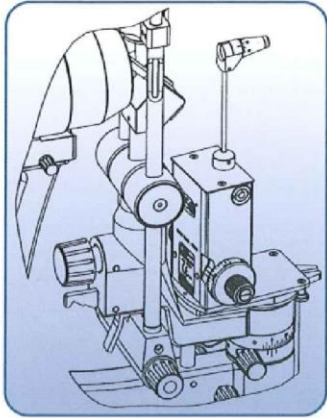
The Z 800 applanation tonometer must be mounted above the microscope of the slit lamp.

The feeler arm protrudes from above into the beam path of the microscope and the illumination unit.

The tonometer is ready for use with a single swinging movement of the feeler arm.

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The instrument F 900 must be placed on the guide plate in one of two possible positions (right or left eye). These positions are related to the microscope, for observation can be made either through the right or the left eyepiece, moreover, the doubling prism, which is held in a ring at the extremity of the pressure arm is automatically in the path of the microscope and the illumination beam.

In order to obtain an image as clear and as free of reflexes as possible, the angle between the illumination and the microscope should be about 60° and the slit diaphragm should be opened completely.

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The instrument A 900 is being produced for those who wish the tonometer to remain permanently on the slit lamp.

It is mounted on a pivot on the microscope and for examination is swung forward in front of the microscope.

A notch position ensures exact centering of the prism with the left objective, in fact the observation of the flattened area of the cornea is made through the left eyepiece only.

When not in use the instrument is secured in a notch position to the right of the microscope.

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Mounting on slit lamps accessories

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Z 800 applanation tonometer

The applanation tonometer Z 800 is an accessory appliance to the SL 980 slit lamps, it's used to measure endocular pressure. Thanks to its versatility the instrument can also be mounted and combined with other equipment. The Z 800 applanation tonometer can also be used as an accessory appliance for slit lamps produced by other manufacturers.

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Applanation tonometers standard accessories

- Measurement head (pressure cone)
code 10.20.01.100,
- Control and calibration lever,
- 2 mm. Allen wrench,
- Assembly plate

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The diagnostic functionalities of the appanation tonometers are constantly developed thanks to the collaboration from surgical and microsurgical divisions of the greatest hospitals, medical institutions, clinics and eye centres.

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© C.S.O. srl 2004 I marchi citati appartengono ai rispettivi proprietari - All trademarks mentioned are of their respective companies
C.S.O. S.r.l. Costruzione Strumenti Oftalmici via degli Stagnacci,12/E